


FRONT CUTTING DRILL

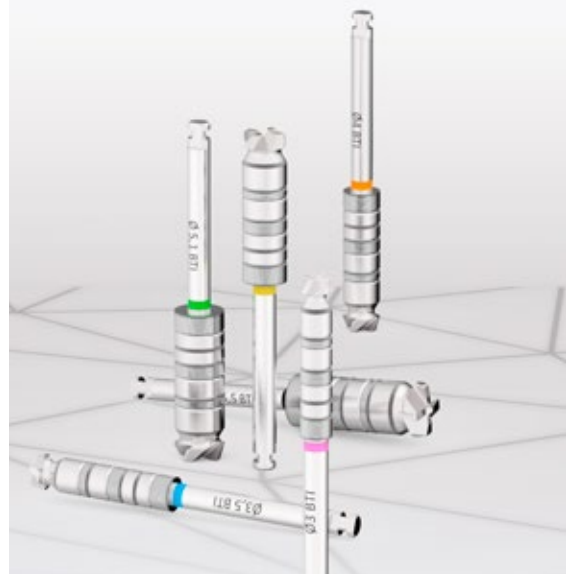

bti.[®]

Human
Technology



Human Technology


FRONT CUTTING DRILL

A CLINICAL DRILLING TECHNIQUE

Front cutting drill, designed to achieve maximum settlement for extrashort implants and to work the cortical bone in transalveolar sinus elevations and in proximity to the dental nerve.


They come in six diameters to be used in the appropriate drilling procedure depending on the implant diameter. The different depth marks are useful to know the exact location of the drill in accordance with the height of the remaining bone.

AN EXCLUSIVE AND INNOVATIVE BTI DESIGN

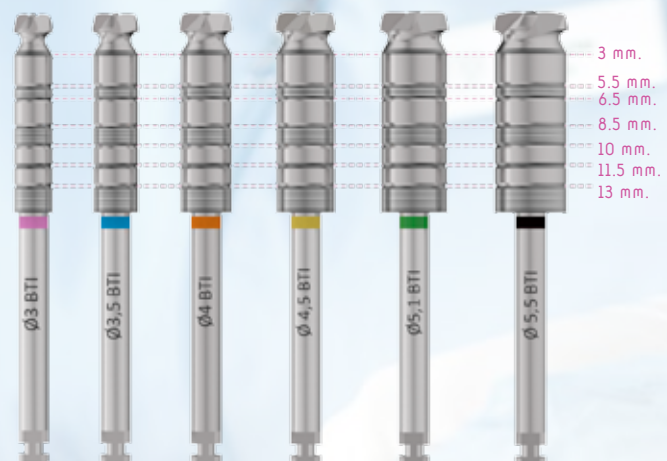


DRILLS AVAILABLE

A CUTTING TOOL THAT DOES NOT DAMAGE ANATOMICAL STRUCTURES

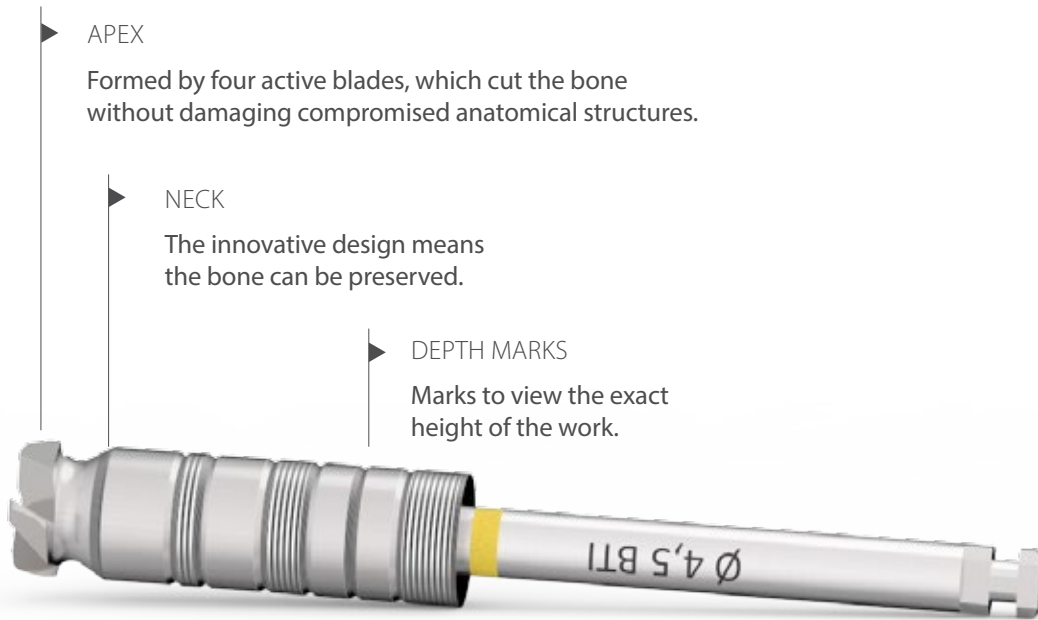


DRILLING PROCEDURE

These drills must be used when the appropriate alveolus for the diameter has been prepared, with apical instrumentation at the base of the sinus and/or allowing further advancement of the extrashort implant, achieving excellent primary stability and avoiding compression.

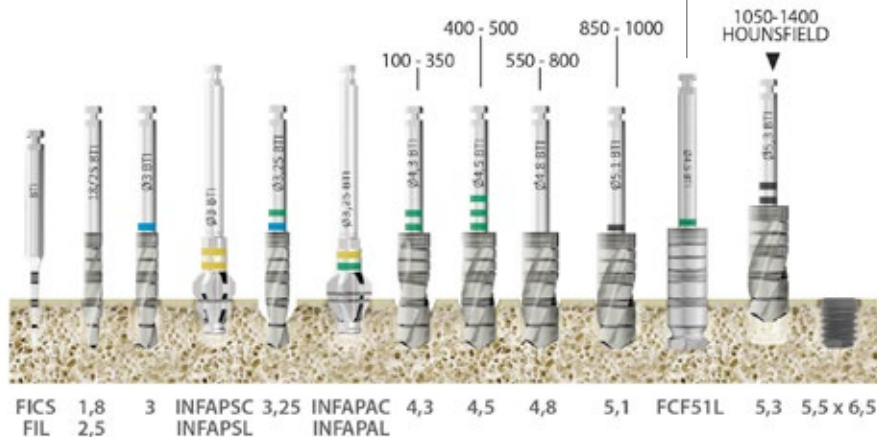


AN IDEAL BTI SOLUTION FOR COMPLICATED SURGERY

▶ TRANSALVEOLAR SINUS ELEVATION

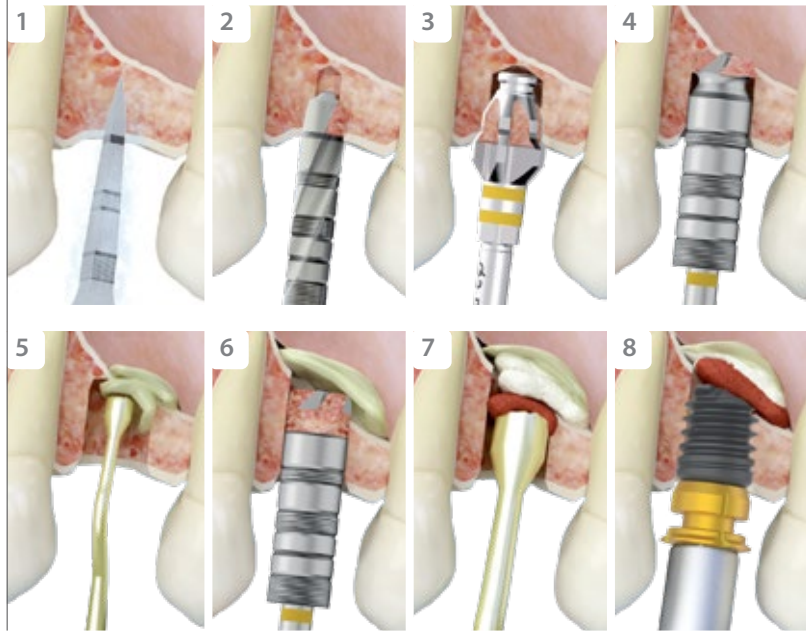

1. Initial drill respecting a 1.5 mm safety margin
2. The drilling diameter is increased in accordance with the choice of implant.
3. Countersink drills to avoid compression at the cortical level. Depending on the final implant diameter, the universal plus drill will be used alone or together with the wide drill.
4. Front cutting drill, wearing down the sinus cortical bone to make a small hole to insert the graft material.
5. Insertion of a Endoret® (PRGF®) fibrin membrane inside the sinus using a bone compactor, before continuing to open the cortical bone in order to detach the Schneider membrane.
6. Complete opening of the crestal window using the drill, with no risk of damaging the sinus membrane.
7. Insertion of graft material (autologous and biomaterial) inside the sinus, until the desired diameter is achieved to insert implants.
8. Insertion of the implant in the prepared alveolus, supported on the sinus cortical and with the apex inside and surrounded by graft material.

▶ PLACEMENT OF SHORT IMPLANTS IN THE JAW

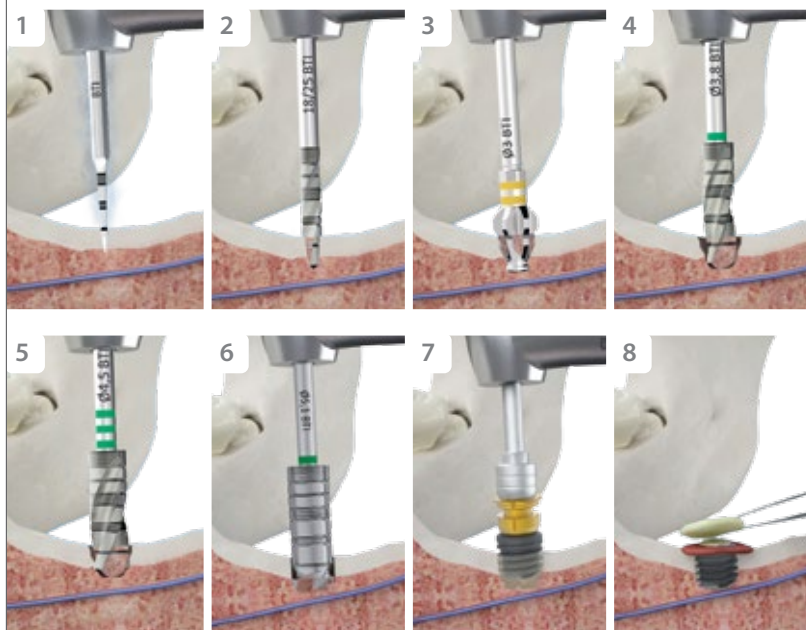

1. Initial drill respecting a 1.5 mm safety margin
2. Drilling procedure to prepare the alveolus, to the same depth as the implant length.
3. Countersink drills to avoid compression at the cortical level. Depending on the final implant diameter, the universal plus drill will be used alone or together with the wide drill.
4. The drilling procedure is continued, gradually increasing the diameter and length of the alveolus.
5. The last drill before the implant must have the right diameter to achieve primary stability, always avoiding compression.
6. Front cutting drill, working in the apical area of the alveolus to achieve implant settlement without compression at that level. Sometimes it is necessary to work on the upper cortical bone of the canal.
7. Insertion of the implant with the surface moistened with PRGF®-Endoret® in the moistened alveolus to the desired level.
8. Supracrestal implant in two surgical phases covered with graft material + Endoret® (PRGF®) to generate vertical growth around it.



COMPETITIVE ADVANTAGES

COLLECTION OF AUTOLOGOUS BONE



The detached bone particles from the drilling build up in the spaces between the cutting blades and are displaced to the retention zone.



THE BEST INSTRUMENT TO AVOID COMPROMISED SURGERY

These drills allow transalveolar sinus elevation via a minimally invasive technique.

They allow extrashort implants to be placed near the dental nerve with excellent control, reducing the risk of damaging it.



bti[®]

Biotechnology
Institute
Human Technology



BTI Comercial

San Antonio, 15 · 5º
01005 Vitoria-Gasteiz
(Álava) · SPAIN
Tel: +34 945 140 024
Fax: +34 945 135 203
pedidos@bticomercial.com

Subsidiaries

GERMANY

Mannheimer Str. 17
75179 Pforzheim · Germany
Tel. +49 (0) 7231 428060
Fax +49 (0) 7231 4280615
info@bti-implant.de

FRANCE

6 Avenue Neil Armstrong
Immeuble Le Lindbergh
33692 Merignac CEDEX · France
Tel: (33) 06 45 07 36 78
info@bti-implant.fr

ITALY

Piazzale Piola, 1
20131 Milano · Italy
Tel: (39) 02 7060 5067
Fax: (39) 02 7063 9876
bti.italia@bti-implant.it

MEXICO

Ejército Nacional Mexicano 351, 3A
Col. Granada Delegación Miguel Hidalgo
Messico DF · CP 11520 · Mexico
Tel: (52) 55 52502964
Fax: (52) 55 55319327
bti.mexico@bti-implant.com

B.T.I.

Biotechnology Institute S.L.

Parque Tecnológico de Álava
Leonardo da Vinci, 14
01510 Miñano (Álava) SPAIN
bti.implantes@bti-implant.es

PORTUGAL

Praça Mouzinho de Albuquerque, 113, 5º
4100-359 Porto · Portugal
Tel: (351) 22 120 1373
Fax: (351) 22 120 1311
bti.portugal@bticomercial.com

UK

870 The Crescent
Colchester Business Park · Colchester
Essex CO49YQ · UK
Tel: (44) 01206580160
Fax: (44) 01206580161
info@bti-implant.co.uk

USA

1730 Walton Road
Suite 110
Blue Bell, PA 19422-1802 · USA
Tel: (1) 215 646 4067
Fax: (1) 215 646 4066
info@bti-implant.us

Distributors

CROATIA

Tel: 00385-1-58-013-02
bti@ortho-plan.com
www.ortho-plan.com

GREECE

Tel: +30 210 9765002
abalase@otenet.gr
www.abalase.gr

JAPAN

Tel: 81-(0)3-3833-3986
ueki@mu-trade.com

LITHUANIA

Tel. +370 5 2136648
Tel. +370 652 29991
info@mpdental.lt

ROMANIA

Tel:(40)769 059 900
office@dent-a-america2000.ro
www.dent-a-america2000.ro

RUSSIA

Tel.: 0074957672311
s.krikorian@surgitech.ru
www.surgitech.ru

UKRAINE

Tel. +380 (73) 031 31 31
info@biotechmedical.com.ua
www.biotechmedical.com.ua

BELARUS

Tel.: (+8) 232 41 00 36
ulibka132@rambler.ru
www.smile132.by

COLOMBIA

Tel: +57 (316) 742-6872
viglesias@rocol.com.co
www.rocol.com.co

MORROCO

Tel.: +212 661 880 824
innovatedentalmaroc@gmail.com

TAIWAN

Tel: +886-2-2647-9077
kevin_lin@smart-linkx.com.tw

www.bti-biotechnologyinstitute.uk

NOTE: consult your distributor for the availability of the product in the different markets.



BTI APP
NEW PRODUCTS

iPhone / smartphone version
iPad / Tablets version (Customer Area)